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DURASOFT® – Hints for Use

Properties:

DURASOFT® is a top quality compound material fabricated by co-extrusion which consists of a hard polycarbonate (PC) base material and a soft (PU) material. A homogeneous surface of the two materials is guaranteed if used correctly. The material is abrasion resistant to the greatest possible extent and almost unbreakable. It is available in 1,8 mm (0,8 mm hard PC / 1,0 mm soft PU) as well as in 2,5 mm (1,1 mm hard PC / 1,4 mm soft PU).

Like all SCHEU-DENTAL pressure moulding materials DURASOFT® was tested at the hygienic institute of the University of Heidelberg for toxicity, irritation of the gingiva and mutagenity. The material was found to be **biologically well-accepted**.

Application:

Inserting and wearing the splint is more comfortable for the patient due to the soft flexible layer, therefore DURASOFT® is well suited for all splint therapies as well as partially for mouth protection and intraoral snoring devices. Regarding orthodontic appliances (application) also slight single-tooth movements are possible due to the soft PU, thus enabling the fabrication of SetUp splints.

Predrying:

**WE RECOMMEND:
Drying Oven**

As a lot of high quality thermoplastics, especially polycarbonate material, DURASOFT® is **hygroscopic** reactive, i.e. absorbs humidity, so that **bubbles** cannot be excluded when the material is heated. To avoid this, the material has to be predried in a BIOSTAR® / MINISTAR®-Drying Oven (REF 3129) or any similar oven at a temperature of approximately **80 °C for at least 4 hours and 8 hours at most**, depending on the strength of the material. If the material has been once predried, it can be used for approximately one week without further pretreatment. Storing the material in the drying oven for **more than 8 hours is not recommended** as the material may change colour and become **yellow**.

Pressurizing:

**WE RECOMMEND:
ISOFOLAN®**

The model should be **insulated** with an ISOFOLAN®-foil (REF 3207) in order to receive perfect **transparency** and smooth inner surfaces. Cut the ISOFOLAN®-foil to the desired shape of the splint and perforate interdentially/occlusally with a probe/scalpel to avoid air bubbles. Now the insulated model is being embedded into **pellets** and coated with DURASOFT®. For code and heating time of Durasoft please refer to the imprint on each plate or the label.

Adjusting:

**WE RECOMMEND:
PERLIBOND**

If an **adjusted bite splint** is required, **cold cure resin**, e.g. DURASPLINT®, can be added to the hard layer (PC). To avoid thermal stress / tensions the moulded splint is not yet finally finished; first the occlusal surface has to be prepared twice with the bonding agent PERLIBOND (No. 4239) and the resin has to be applied on the moist bonding agent. Polymerisation is done according to the instructions of the supplier, only after this process the splint is finally finished and removed.

Finishing:

**WE RECOMMEND:
Finishing-Set
DIMO®**

For final finishing and trimming, we recommend the finishing set (REF 3378). The set provides for example the HM-carbide cutter (REF 3369) and the cutting bur (REF 3214) for coarse cutting of splints. For fine interdental finishing of splints, we recommend the HM-tricutter (REF 3370). Final trimming and polishing can be most effectively done with the POLYFIX black (REF 3371) or the OSAMU-polisher (REF 3247).

For final trimming and polishing of hard/soft foils as well as for final finishing of the soft splint parts, DIMO® and DIMO®PRO abrasives (REF 3380–3384) can be recommended as well.

Material Care:

**WE RECOMMEND:
CETRON®**

Please advise your patients to clean the splint carefully. We recommend daily cleaning with our CETRON® products. Alternatively, splints can be cleaned with tooth brush, toothpaste and clear water/mouthwash. In order to avoid modifications and staining, please don't use no cleansing agents containing colorings. Please don't use neither steam blasting and ultrasonic devices, nor prosthetics cleansing agents or any cleaning method with a temperature higher than 70°C .

Due to contact with individual mouth environment, lack of hygiene, strong aromatic food and drinks such as nicotine, coffee, tea, coke and pharmaceuticals, splints may change their color, taste and smell.

Moreover, please don't use no aromatic hydrocarbons such as benzol, aliphatic alcohols, toluene, ethanol, acetone etc. for cleaning. Some products on herbal basis used for temporary treatment of algetic and inflammable conditions of the oral mucosa, especially vesicles as well as for symptomatic treatment of gingivitis may lead to strong stainings, maceration and destruction of the resin and should be strictly avoided (f.e. PYRALVEX®).

Stainings may occur, too, in case of interactions of filling materials and prosthetics (f.e. amalgam, molybdenum of chromium cobalt and alloys with a reduced content of precious metals.)

Please contact your dentist immediately in case of any modification of your splint.